# i-DAM

<b>EN</b>	Light curing liquid cofferdam
<b>LT</b>	Šviesoje kietėjantis skystas koferdamas
<b>PL</b>	Światłoutwardzalny ciekły koferdam
<b>RU</b>	Светоотверждаемый жидкий коффердам
<b>DA</b>	Lyshærdende flydende kofferdam
<b>NO</b>	Lysherdende flytende kofferdam
<b>SV</b>	Ljushärdande flytande kofferdam
<b>DE</b>	Lichthärtender flüssiger Kofferdamm

## INSTRUCTION FOR USE

EN

### DESCRIPTION

i-DAM is a light curing liquid cofferdam, indicated as barrier for the isolation of gingival tissues and replacement of the rubber dam sheet to protect the gingival tissue in some cases where there is a risk of irritation by contact with products used in the clinical procedures, such as dental bleaching with Hydrogen Peroxide. The product is sufficiently viscous and allows easy application and control over the place to be protected. It is blue or green in colour in order to make a contrast with the gingival tissues and the teeth and so facilitate its localization.

### INSTRUCTION FOR USE

**NOTE!** Do not use if primary package is broken. It is recommended prepare syringe just before applying.

#### Syringe preparation:

1. Remove Luer cap.
2. Attach working tip to the syringe.
3. Verify resin flow prior to applying. If resistance is met, replace tip and re-check. Use only recommended tips.

**NOTE!** DO NOT APPLY product on gingival tissue in poor conditions, people who have undergone recent periodontal surgery or which contains a previously installed irritation.

1. Clean and dry the gingival tissues and the teeth in the area which you wish to protect.
2. Apply a fine layer (maximum of 1-2mm) of the product on the gingiva and the interdental papilla in a way that they are completely covered and there will be a good field of protection for the procedure to be applied.
3. Light cured for 20-40 seconds. It is advisable to proceed with the polymerization soon after applying the resin in order to prevent any possible irritation of gingiva. During the process of polymerizing a slight sensation of heat or burning may be felt on the gingiva owing to the reaction of the polymerization and contraction of the resin.
4. Verify that the product covers the entire gingiva surface and that it is completely polymerized. In order to see the rear part of the papilla and gingiva, use a clinical mirror looking from incisal to cervical. Avoid testing it by forcing it or trying to lift it as this can cause infiltration of the irritating substance.
5. Do dental bleaching procedures.
6. In order to remove the polymerised product after procedures just move it forwards. It should come loose easily as a whole or in large pieces.

### WARNINGS AND PRECAUTIONS!

**NOTE!** Do not use i-DAM if patient is allergic to product or to any of the ingredients.

#### Hazard and precautionary statements:

In susceptible individuals, product may cause allergic or irritation reactions (skin, eye, mucosa, respiratory tract). Product causes skin or mucosa irritation and serious eye irritation. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention.

#### Prevention:

Wear protective gloves and avoid repeated contact of uncured resin with skin or mucosa. Wear protective clothing, eye and face protection.

#### Interaction with other substances:

Avoid interaction with eugenol, strong bases, strong oxidizing agents.

### STORAGE

Re-cap syringe with cap between uses. Keep product tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not freeze. Keep out of the reach of children! Avoid release to the environment. Dispose of contents/container to as required by national regulatory requirements. Do not use after the expiry date.

### MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

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