Vitique Try-In

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English

Product description

Using the Vitique Try-In paste allows the final shade result in the mouth to be accurately simulated, and the correct shade of Vitique Esthetic Resin Cement or PermaCem Universal to be determined. Water-soluble Vitique Try-In pastes are made to correspond to the shades of hardened dental cements.

Indications

Paste for shade simulation.

Contraindications

Do not use the material where there are known allergies to any of the components or contact allergies.

Basic safety information

- ▶ For dental use only.
- ▶ Keep out of the reach of children.
- ▶ Avoid eye contact. In the event of inadvertent contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician.

Side effects

There are no known side effects to date. Hypersensitivity to components of the material cannot be excluded in individual cases.

Notes for use

The cement shade is selected prior to placing the rubber dam.

Recommended use

1. After removing the temporary, clean the preparation margin with fluoride-free polishing paste; in doing so, remove residual temporary luting material, rinse, and dry with oil-free air.
2. Try in the restoration.
3. Apply Vitique Try-In paste to the inside of the restoration.
4. Insert the restoration and check the selected shade.
5. If the selected shade of cement is being corrected, the stump and restoration are to be cleaned with water and the try-in is to be repeated with a suitable shade.
6. Remove the restoration.
7. Clean the restoration and preparation margin using a water spray and dry with air that is free from oil and water.

Note: Any residues of Vitique Try-In Paste can influence the permanent cementation of the restoration.

8. Silanize silicate ceramic again if necessary (e.g. with Vitique Silane).

Storage and shelf life

- ▶ Store in a dry place (15 – 25 °C/59 – 77 °F) at room temperature.
- ▶ Do not use after the expiration date.
- ▶ Close packaging immediately after use.

Composition

Pyrogenic silicic acid, pigments in a water-soluble polyglycol matrix

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