VOCO Structur 3

EN Instructions for use MD EU Medical device

Product description:

Structur 3 is a self-curing paste-paste system based on multifunctional methacrylic acid esters for the fabrication of temporary restorations.

Shades:

Structur 3 in the cartridge: A1, A2, A3, A3.5, B1, B3, C2, BL

Structur 3 in the QuickMix syringe: A1, A2, A3, B1

Indications:

- Fabrication of temporary crowns, bridges, inlays, onlays, partial crowns, veneers and temporary posts
- Fabrication of long-term temporary restorations
- Rebasing of prefabricated temporary crowns made of composite, polycarbonate or metal

Contraindications:

Structur 3 contains methacrylates, amines, terpenes, benzoyl peroxide, BHT. **Structur 3** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:

Structur 3 is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Structur 3 should only be used by a professionally trained dental practitioner.

Use:

Preparation:

Structur 3 in the cartridge:

Structur 3 cartridge into the **VOCO** dispenser (type 2) or a comparable dispenser. For technical reasons, there may be slight differences in the filling levels of the two cartridge chambers prior to initial use. In this case, remove the cartridge cap and carefully squeeze the material out until it is delivered evenly from both openings. Then attach a type 6 mixing tip and lock with a 90° turn clockwise. The material is extruded by pressing the lever and is automatically mixed in the correct ratio.

Structur 3 in the QuickMix syringe:

Due to technical reasons slight differences in the filling level between the syringe chambers can occur prior to the first use. Therefore remove syringe cap and squeeze the material out until the material is delivered evenly from both chambers. Afterwards place a mixing tip type 10. The slots on the syringe and the mixing tip must correspond. The mixing tip is fixed by rotating it clockwise through 90°. The material is automatically mixed in the correct ratio in the tip as it is dispensed and can be applied immediately. All QM syringes come with a detachable piston. The piston can easily be pulled backwards out of the syringe and then reinserted. The seals in the syringe do not change their position whilst this is being done. This prevents cross-contamination of the material in the syringe.

The warranted product characteristics can only be achieved when using the (original Sulzer Mixpac) mixing tips as approved by VOCO and available as accessories.

The attached mixing tip is intended for single use only. After use, store the cartridge / QM syringe tightly sealed. Check that material flows freely through the openings before reuse.

Processing directions:

First level out any undercuts in the preparation using a non-eugenol cement. Take an impression if not using prefabricated dental copings or similar restorative materials.

Prepare the impression by levelling undercuts and cutting drainage channels. If the wall thickness of the temporary restoration is expected to be too thin, enlarge the impression, paying particular attention to removing the septums in the interdental embrasures. In case of gaps in the molar region, cut a groove in the impression to maintain a stable, bar-shaped connection in the temporary restoration.

Then clean and air dry the impression.

Apply **Structur 3** directly into the impression, applying the material initially to the occlusal surfaces of the impression and filling up towards the gingival areas. To prevent the formation of bubbles during application, always keep the opening of the mixing tip immersed in the material. Insert the impression into the mouth no later than 30 seconds after commencement of mixing.

Remove the temporary restoration from the mouth during the elastic phase i.e. within 1 to 1.5 minutes after beginning to mix.

The setting process should additionally be monitored by observing the excess material in the mouth.

The setting time of material (in mixing tips) that has not been used cannot be compared to the setting time of the material in the mouth.

The working times at a room temperature of 23 °C and 50 % humidity are as follows:

min:s	min:s	min:s	min:s	min:s
0:00	0:30	1:00	1:30	4:00
insertion the mouth	setting in the mouth	removal	complete setting	finishing

Different room temperatures may result in variations from the above times.

Higher temperatures shorten and lower temperatures extend these times.

Finishing:

Extract the temporary restoration from the impression after 4 minutes at the earliest and completely remove the inhibition layer, caused by oxygen in the air, by wiping with alcohol (e.g. ethanol).

Then carefully grind any excess material and rough areas near the gingiva.

If necessary, finish the temporary restoration with fine, cross-cut carbide burs. If required, polish the temporary restoration manually (e.g. with a rubber polisher, fine sandpaper or goat hair brush).

Luting:

Carefully clean and dry the core, if necessary removing any insulating material. Attach the finished temporary restorations with a temporary luting cement.

Eugenol cements can inhibit the setting of luting composites that may be used later at the permanent luting stage. If a luting composite is to be used for permanent luting, attach the temporary restoration with a eugenol-free cement.

Customisation, repairs and rebasing:

The following procedures are recommended for customisation, repairs or rebasing:

Repair of defective temporary restorations:

- **Air bubbles:** fill with **Structur 3**.
- **In the case of a fracture of a newly made temporary restoration:** re-connect the fracture with freshly mixed **Structur 3**.
- **Repair of temporary restorations that have already been worn:** Roughen the fracture with a bur and place undercuts. Apply bond (e.g. **Futurabond U**) to the prepared fractures and cure according to the respective instructions for use. Apply a flowable material to the prepared areas (**GrandioSO Flow** is suitable for this purpose because it provides optimum shade matching). Reassemble the fractured pieces and cure according to the instructions for use.

Customisation:

Apply a flowable material (e.g. **GrandioSO Flow**) in the required shade to the freshly prepared temporary restoration and then cure and finish according to the instructions for use.

Rebasing prefabricated, temporary crowns:

- Apply **Structur 3** to the prefabricated, temporary crown, following the respective manufacturer's instructions for use.
- Place the prefabricated crown on the prepared core.
- Remove the prefabricated crown from the mouth during the elastic phase, i.e. 1 to 1.5 minutes after commencement of mixing.
- In addition, monitor the setting process by observing the excess material in the mouth. The setting time of material (in mixing tips) that has not been used **cannot** be compared to the setting time in the mouth.
- Remove excess material after 4 minutes at the earliest and finish.

Warnings, precautionary measures:

- Do not inhale the dust created during the finishing of the temporary restoration. Wear a mask and protective goggles and use an extraction system.
- Before the temporary restorations are made, the prepared cores can be insulated to prevent hypersensitivity (by using fluoride varnishes, core varnishes and the like).
- Ensure that methacrylate-based linings and core build-ups (e.g. composites, compomers, ORMOCER®, resin-reinforced glass ionomer cement) are adequately insulated (e.g. with glycerine) to prevent the temporary restoration becoming bonded to the core build-up.
- Inadequate hygiene or the intensive use of certain mouth rinses may lead to shade variations.
- Do not start to finish the temporary restoration until the material has completely set (approximately 4 minutes after commencement of mixing).
- If necessary, (in the case of a long-term temporary restoration) the temporary restoration should be checked and, where required, replaced.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Storage:

Structur 3 Store cartridges protected from light i.e. in closed packages, drawers etc. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Structur 3** must be reported to VOCO GmbH and the responsible authority.

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VC 60 BB2500 E1 0721 99 © by VOCO
ORMOCER® is a registered trademark of Fraunhofer Gesellschaft zur Förderung der angewandten Forschung e.V., München, Germany.