

COMPLIANCE ENABLING ENVIRONMENTAL DISINFECTION

Incidin™ OxyWipe S

SPORICIDAL (*C. DIFFICILE*) READY-TO-USE SURFACE CLEANING AND DISINFECTION WIPES

The advanced formulation of Incidin™ OxyWipe S provides the benefits of a patented* hydrogen peroxide solution with broad spectrum efficacy. It is effective against a wide range of pathogens and leaves no active residues.

- Active ingredient decomposing into water and oxygen
- Sporicidal (*C. difficile*) & virucidal offering
- Excellent material compatibility
- Incidin OxyWipe S efficacy confirmed for liquid released from the wipe



*European Patent EP1987121

Incidin™ OxyWipe S

SPORICIDAL (*C. DIFFICILE*) READY-TO-USE SURFACE CLEANING AND DISINFECTION WIPES

EFFECTIVE

- Broad spectrum efficacy

SAFE

- Active ingredient decomposes into water and oxygen
- Excellent material compatibility

CONVENIENT

- Cleans and disinfects
- Ready to use formats

MICROBIOLOGICAL PROPERTIES FOR MEDICAL DEVICE APPLICATION

EFFICACY UNDILUTED	CONTACT TIME
Disinfection of surfaces with mechanical action acc. to VAH method 2015	
Bactericidal, yeasticidal	15 min*
Fungicidal	30 min*
Mycobactericidal	30 min*
Bactericidal according to EN 13727 (2min) / EN 16615 (2 min*)	2 min*
Yeasticidal according to EN 13624 (15min) / EN 16615 (1min*)	15 min*
Fungicidal according to EN 13624 (30min) / EN 16615 (method) (5min*)	30 min*
Tuberculocidal according to EN 14348 (30min) / EN 16615 (method) (5min*)	30 min*
Mycobactericidal according to EN 14348 (30min) / EN 16615 (method) (5min*)	30 min*
Effective against enveloped viruses according to EN 14476	30 sec*
Limited spectrum virucidal according to EN 14476 (Adenovirus - 30sec, Norovirus - 15min)	15 min**/**
Virucidal according to EN 14476	30 min**
Sporicidal against <i>C. difficile</i> according to EN 17126	60 min**
Sporicidal against <i>C. difficile</i> according to EN 16615 (method) according to CEN TC216 prWI Oct 2020 / VAH method 19	15 min**
Effective against enveloped viruses according to RKI recommendation 10/2022	30 sec***
Limited spectrum virucidal	60 min***
Adenovirus	1 min***
Polyomavirus SV40	30 sec***
Rotavirus	15 sec***

* dirty conditions ** clean conditions *** DVV 2015 (suspension test)

MEDICAL DEVICE AND BIOCIDES USE SAFELY

Please read label and product information.

APPLICATION AREA



Medical Device: Broad spectrum ready to use cleaning and disinfection wipe for non-invasive, invasive and non immersible medical devices.

Biocide: Disinfection of non-food contact surfaces in health care applications by wiping using impregnated RTU wipes.

INSTRUCTIONS FOR USE

Take single wipe. Wet surface thoroughly, ensuring contact with entire surface. Let it dry and do not rinse. Do not reuse wipe: Disinfection can no longer be guaranteed. Close lid after use. Respect the indicated contact time and conditions for the required antimicrobial activity. If used under clean conditions, clean surface upfront (use new wipe for each step). The use of a cleaning and disinfection wipe does not replace regular cleaning. Do not use on surfaces sensitive to oxidative agents such as marble, copper or brass. For reprocessing of medical devices always observe manufacturers' instructions for use, incl. material compatibility. Terminal disinfection of semi-critical medical devices (i.e. probes & TEE probes): If immersion is not possible, disinfect the device by wiping as described above. Rinse thoroughly with water of drinking quality in this case after contact time elapsed.

INGREDIENTS

100g contains the following active ingredient: 1.5g Hydrogen peroxide (CAS 7722-84-1)

STORAGE

Product should be stored in the original container between 5°C and 25°C in a cool, dry location out of direct sunlight.

Shelf life after first opening: 3 months.



FOR PROFESSIONAL USE ONLY

Incidin™ OxyWipe S

PACKAGING AND ORDERING INFORMATION

PRODUCT	PACKAGING UNIT	ORDER CODE
Incidin™ OxyWipe S	6 x 100 wipes (20 x 20 cm)	3083020
Wall mounted holder for wipe flowpacks (durable Polypropylene)	12 x 1	100 576 86

ECOLAB HEALTHCARE
Winnington Avenue,
Northwich,
Cheshire CW8 4DX
United Kingdom



ECOLAB DEUTSCHLAND GmbH
Ecolab-Allee 1
40789 Monheim am Rhein
Germany

ECOLAB®
PROTECTING WHAT'S VITAL™