**Sendoline S5 System STERILE**  
**Sendoline Big Apical Files**  
**Sendoline S1 System**  
**Sendoline S3 System**

**(EN)**

**Instructions for use**  
**Sendoline sterile rotary and reciprocating endodontic files**

**Intended use**  
Sendoline instruments covered by these instructions are intended for mechanical treatment of the root canal by qualified users in a clinical environment. The purpose of the instruments is to clean and shape the root canal system.

**Design**  
The device operative part is made from nickel-titanium and is defined as a type 2 instrument according to EN ISO 3630-1. The shank is type 1 according to EN ISO 1797.

**Contra-indications**  
General contra-indications for root canal treatment.

**Precautions**  
- Prior to use, store the instruments in their original packaging in a dry, dust free and clean environment. Protect from sunlight.  
- The instruments are provided in sterile condition; sterilized using gamma. If the package is damaged or unintentionally opened, the instruments must not be used.  
- The instruments are intended for single use with the purpose to avoid cross-infection and to minimize the risk of file fracture following repeated use.  
- The instruments are sharp and must be handled with care to avoid injuries. Follow national healthcare regulations for safe disposal of sharps.  
- In case of a serious incident from using the instruments, immediately contact Directa AB and the applicable authority in your country.

**Indications for use**  
The instruments are intended for mechanical instrumentation and should be used together with the specified endodontic handpiece/motor using the settings, if any, specified in the tables in this document.

- Create straight-line access to the canal orifice(s).  
- Establish working length (WL) and create a reproducible glide path with small size manual hand instruments and/or mechanical glide path instruments.  
- Work in the canal using the file sequence of the device model of choice in accordance with canal shaping techniques applicable to mechanical instruments.

**Symbols**

<b>R<sub>only</sub></b> USA: For professional use only	 Reciprocating movement
 Setting	 Clockwise rotation
 File type S – the S figurative mark is the property of Directa	 File type K

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Table with 2 main columns: S5 Endo Motor and Sendoline Endo Motor. Rows include #1 30/08, #2 30/06, #3 30/04, #4 25/04, #5 20/04. Columns show rotation direction, Ncm, and r/m.

Table with 2 main columns: S5 Endo Motor and Sendoline Endo Motor. Rows include 35/04, 45/04, 40/04, 50/04. Columns show rotation direction, Ncm, and r/m.

Table with 2 main columns: Sendoline S3 System and Sendoline Endo Motor. Rows include #1 30/08, #2 30/06, #3 30/04, #2 30/06, Glidepath 20/04. Columns show rotation direction, Ncm, and r/m.