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EN	Foam for staunching bleeding
BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY TO BE USED BY DENTIST ONLY	

COMPOSITION: aluminium chloride 20%, water Ph. Eur., excipients.

INTENDED USE
ALUSTAT FOAM is intended for use during dental treatment to staunch slight gingival bleedings.
Aluminium chloride in the product has the vasoconstrictive effect on tiny blood vessels, as a result of which the gingival swelling and slight bleedings decrease.

INSTRUCTIONS FOR USE

Before the treatment, immerse the supplied applicator in alcohol, allow it to dry an then install the applicator on the syringe.

To stop the bleeding:
Apply ALUSTAT FOAM directly on the bleeding spot. Rinse after 2 minutes. Repeat if necessary.

To retract the tissues:
1. Apply ALUSTAT FOAM into gingival sulcus using the supplied applicator. Rinse after 2 minutes or

2. Insert the retraction cord into gingival sulcus and place ALUSTAT FOAM on the cord. Rinse after 2 minutes.

A reusable product, use with a single-use applicator. Re-using the applicator may cause the risk of reinfection.

CONTRAINDICATIONS
Do not use for patients who are hypersensitive to the product components.

PRECAUTIONS

Before taking an impression, ensure that you remove the preparation to avoid inhibiting the polymerisation of the impression material. Aluminium chloride can adversely affect the bonding strength of dentin to tooth restoration materials. ALUSTAT FOAM has cytotoxic potential. The use of products with aluminium chloride contents may cause local and temporary tissue damage.

Irritating to the mucous membranes of the oral cavity, skin and eyes.
In case of contact, rinse immediately with plenty of water and seek medical advice.
If swallowed, do not induce vomiting.
Drink plenty of water.
Seek medical advice. Use a cofferdam, disposable gloves, safety goggles/visors and protective clothing when handling the product. Keep out of reach of children.
Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE
Keep in original packaging below 25°C.
Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.
PACKAGING
Syringe 0,8 g with preparation, disposable applicators.
Last update: 27.02.2025/10

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