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**Correction impression**

- Fill the prepared impression tray with Silagum-MixStar Putty Soft. When doing so, hold the impression tray at a slight angle beneath the mixing tip.

**Note:** To avoid air bubbles, the end of the mixing tip should stay immersed in the impression material throughout the application process.

- Insert the tray in the mouth within the working time. Maintain this position in accordance with the recommended intraoral setting time.
- Remove the impression from the mouth once the setting time has ended.
- Cut generous drainage channels into the pre-impression for the wash and remove imperfections.
- After repositioning, thoroughly rinse the impression with water and dry.
- Apply Silagum-Light around the preparation and on the pre-impression.
- Position the filled tray in the mouth within the working time and initially fix in place by applying even pressure.
- Remove the impression from the mouth once the setting time has ended.

**Removal of the impression**

- Remove the impression parallel to the longitudinal axis of the prepared teeth.
- After removal, proceed as described in the section "Cleaning and disinfection of the final impression".

**Cleaning and disinfection of the final impression**

- Only disinfect the final impression using dipping solutions. DMG recommends the use of a 2% glutaraldehyde solution.
- Please ensure that the disinfectant used is indicated for use with addition curing silicone impression materials.
- Follow the instructions of the disinfectant solution's manufacturer precisely.
- Leave the final impression to dry and store until casting, see "Storage of the impression".
- For more information, refer to the comments of the DGZMK. Comments of the DGZMK 3/93 V 1.0, www.dgzmk.de.
- ADA recommends following Council on Dental Materials, Instruments and Equipment. Disinfection of impressions. JADA 1991;122(3):110; www.ada.org

**Casting and galvanization**

- The impression should be cast 30 minutes at the earliest after removal from the mouth.

DMG recommends using special purpose hard plaster (ISO 6873; Type 3, e.g. Moldano, Bayer) for the cast and an ultra hard dental plaster for the forms (ISO 6873; Type 4, e.g. Fujirock II, GC). The impression can be copper-plated or silver-plated.

**Storage of the impression**

- Store impression at room temperature (15–25 °C/59–77 °F) in a dry place!
- Protect against exposure to direct sunlight!

The impression is stable and can be cast even after months without loss of accuracy.

**Technical data**

Working time*	≤ 2:00 min
Recommended time in mouth	≥ 3:30 min
Dimensional change after 24 h	≤ 0,5%
Compression set	≤ 0,5%
Strain in compression	1,7–2,6%

\*The times apply at a room temperature of 23 °C and a normal relative air humidity of 50%. Higher temperatures shorten, lower temperatures increase this time.

**Storage and shelf life**

- Store at room temperature (15–25 °C/59–77 °F) in a dry place.
- Do not store in the fridge or freezer.
- Protect against exposure to direct sunlight.
- After the application, leave the used tip on the cartridge to cap unused material.
- Do not use after the expiration date.

**Composition**

Vinyl polysiloxanes, hydrogen polysiloxanes, fillers, pigments, additives, platinum catalyst

**Classification**

ISO 4823:2000, type 1: Heavy bodied consistency

**Available product sizes**

1 cartridge @ 380 ml paste, 10 mixing tips	REF 909534
5 cartridges @ 380 ml paste, 50 mixing tips	REF 909535

Federal law (USA) restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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