NeoPUTTY® Non-Staining BIOACTIVE Bioceramic

AVALON BIOMED

Advanced Bioceramics

INSTRUCTIONS FOR USE

NeoPUTTY®

COMPOSITION & DESCRIPTION


Multi-purpose root and pulp treatment bioactive paste consisting of an extremely fine, inorganic powder of tricalcium/dicalcium silicate in an organic medium. The product is packaged ready-to-use. No mixing is required. NeoPUTTY is designed to set in vivo in the presence of moisture provided by the surrounding tissues.

Immediately **restore** over NeoPUTTY with a light curable composite, glass ionomer, RMGI, compomer, or luting cement and crown.

Alternatively, you may use a flowable composite, RMGI, ZOE or other material **to secure** the NeoPUTTY prior to final tooth restoration.

g) Assess the pulp vitality as needed and confirm with a radiograph.

PULPOTOMY and APEXOGENESIS: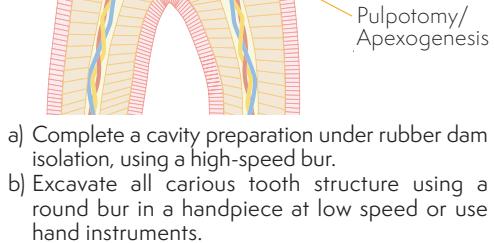

a) Complete a cavity preparation under rubber dam isolation.

b) Excavate all carious tooth structure using a round bur in a handpiece at low speed or use hand instruments.

c) In multi-rooted teeth, remove the roof of the pulp chamber and all remnants of coronal pulp tissue to the level of the orifice of each root canal.

d) In single-rooted teeth, remove the pulp to the level of the cemento-enamel junction or slightly below.

e) Control hemorrhage using a solution of your choice (e.g. sterile saline, sodium hypochlorite (1.25-6.0%) or chlorhexidine). If hemorrhage is still present after 10 minutes, the diagnosis is irreversible pulpitis and a full pulpotomy with obturation is typically performed instead.

f) Use applicator of your choice to apply NeoPUTTY material on the exposed pulp or the floor of the cavity preparation, maintaining a minimum thickness of 1.5 mm.


g) Excess material may be removed using a cotton pellet slightly dampened with sterile water or saline.

f) NeoPUTTY is washout resistant when placed. Immediately **restore** over NeoPUTTY with a light curable composite, glass ionomer, RMGI, compomer, or luting cement and crown.

Alternatively, you may use a flowable composite, RMGI, ZOE or other material **to secure** the NeoPUTTY prior to final tooth restoration.

g) Assess the pulp vitality as needed and confirm with a radiograph.

PERFORATION REPAIR, RESORPTION or APEXIFICATION:

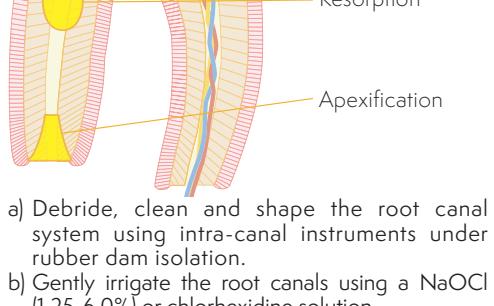a) Isolate the defect site(s).

b) Obtain the canal space apical to the defect.

c) Dispense NeoPUTTY material into the defect site with an instrument of clinician's choice.

f) Gently compact NeoPUTTY material using a small plugger, cotton pellets or paper points.

g) Confirm placement with a radiograph.

h) Excess material may be removed using a cotton pellet damped with sterile water or saline.

i) Obtain the remaining canal space and close the coronal access.

FOR APEXIFICATION:

c) Dry the canal system with paper points, being careful not to extend the points beyond a wide-open apex.

d) Gently compact NeoPUTTY in the apical region, to create a 3 to 5 mm apical barrier.

e) Confirm placement with a radiograph.

f) Rinse gently.

g) Confirm placement with a radiograph.

i) Close the surgical site.

OTHER APPLICATIONS:

NeoPUTTY may be used with or without NeoSEALER® Flo for complete endodontic obturation when applicable.

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