PL	Silan metakrylanowy
----	---------------------

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EN	Methacrylate silane
BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY TO BE USED BY DENTIST ONLY	

COMPOSITION: methacrylate silane, organic solvent.

INTENDED USE
SILANE is applied on porcelain after etching, before using bonding system in order to increase the durability of bonding between resin and porcelain.
The product can be used for silanizing the glass fibres.

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CONTRAINDICATIONS
Do not use the product for patients with hypersensitivity to the product's components.

PRECAUTIONS
Use a rubber clam and protection for patient's eyes when repairing teeth in the patient's oral cavity. Use a vacuum system while etching. Equip staff with protective clothing, gloves and visors. The product may irritate the skin, eyes and the respiratory system. In case of eye contact, rinse with plenty of water and contact a doctor. In case of contact with the skin, wash it out with soap and plenty of water. If swallowed, do not induce vomiting. Rinse mouth with water. Contact a doctor.
Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE
Store in original packaging below 25°C.
Keep out of reach of children.
The expiry date is on the direct packaging.
Once container is open, the expiry date doesn't change if the container is securely closed after each use.

TRANSPORT
The manufacturer allows transport of the product for a period not longer than 10 days at a temperature not exceeding 40°C. Short-term transport within the given temperature range does not affect the safety and efficacy of the product throughout its shelf life.

HANDLING THE PRODUCT CONTAINER
Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING
A syringe containing 2 ml of the formulation.

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